






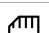
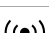
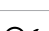








Icon Guide

English

Icon	Description
Diagnostic test status¹	
	Patient undergoing home sleep test.
	Diagnostic data available for physician to review.
	Diagnostic data reviewed by the physician; electronically-signed report is available.
	Interpretation has been completed and awaiting prescription for therapy. ²
	Prescription completed and can be on several stages (Ready to send, Results sent, Patient accepted, or Patient returned). ²
Monitoring type	
	Referred diagnostic patient awaiting setup ²
	Patient not wirelessly monitored
	Data downloaded via SD card
	Patient wirelessly monitored
	Remote Assist only ²
Dashboard notification	
	Patient's prescription changed
	Notes added to a patient file
	Diagnostic report ²
	Prescription for therapy report ²
Patient compliance status	
	Compliance conditions met
	Compliance conditions not met

Icon	Description	
Daily compliance status		
Icon legend:		
	Top—AHI / Bottom—Leak	
	- Below threshold (green)	
	- Above threshold (yellow)	
	- Data not supported (white)	
	Overall colour—Usage	
	- Above threshold (green)	
	- Below threshold (red)	
Usage above threshold	Usage below threshold	
		
		High leak
		Leak data not supported
		High AHI
		AHI data not supported
		High AHI, leak data not supported
		High leak, AHI data not supported
		High AHI, high leak
		AHI, leak data not supported
		Device not used
		Monitoring—no data received yet
		Not monitoring—no data available
		Device has fault—no data available

¹ If a patient has not moved to the next status after 7 days, the icon will change into red. When the next step is completed, the icon will change back to green.

² Feature not available in all regions.



 **ResMed Ltd**
1 Elizabeth Macarthur Drive
Bella Vista NSW 2153 Australia

DISTRIBUTED BY
ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA
EC REP ResMed (UK) Ltd 96 Jubilee Ave Milton Park Abingdon
Oxfordshire OX14 4RW UK

See www.resmed.com for other ResMed locations worldwide.
For patent and other intellectual information, see www.resmed.com/ip
AirVlew is a trademark of ResMed Ltd.
© 2015 ResMed Ltd. 228786/5 2015-12

ResMed.com